

CLÉMENT

RULER

This ruler will allow you to take heights easily and effectively. It has been entirely designed and manufactured in Alsace, and follows our decision in 2020 to stop using presentation glasses

USER MANUAL

1. Place the lower part of the ruler at the bottom of the eyewear, with the high tabs on each side of the frame.
2. Move the ruler horizontally to align its center with the wearer's pupil.
3. Record the height of the pupil.

For any advice, do not hesitate to contact our workshops: order@clementlunetier.com or +33(0)3.88.35.74.71.

